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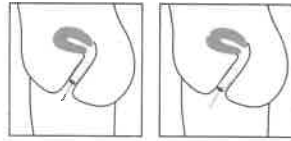
Produced in EU for Creative Concept Nordic AB, SE-183 79 Täby www.creativeconcept.se

EN

topcare® tampons are comfortable to use and provide a safe and secure protection during menstruation. The tampon's highly absorbent core quickly and effectively absorbs liquid, reducing the risk of leaks. The soft, smooth surface layer makes the tampon easy to insert and remove.

Always choose the tampon size that matches your needs.

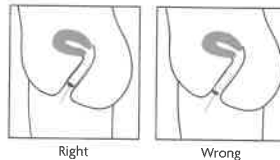
topcare® **MINI** for light bleeding
topcare® **NORMAL** for light to moderate bleeding
topcare® **SUPER** for heavy bleeding

HYGIENE

Good hygiene is important during menstruation. Always wash your hands before and after inserting or removing the tampon. Avoid leaving the tampon in for too long. Change approximately every 4 hours. If you have heavy bleeding, you may need to change more frequently.

USAGE

Remove the tampon's protective covering and check that the string is securely attached by gently tugging on it. Squat down or stand with one foot on the toilet seat. Relax and gently insert the tampon backwards and upwards into the vagina. If you can still feel the tampon, it's not inserted far enough. Gently push it up a little further. To remove the tampon, slowly pull the string forwards and downwards. The tampon and its protective covering should not be flushed down the toilet.

IMPORTANT INFORMATION ABOUT TSS

Toxic Shock Syndrome (TSS), also known as tampon disease, is caused by a toxin that can be produced by certain strains of Staphylococcus aureus bacteria. Symptoms of TSS include fever, fainting spells, dizziness, vomiting, and diarrhea. It is also common to experience redness in the eyes and mouth, rash, and muscle and joint pain. Seek emergency medical attention if you experience these symptoms as the illness is very dangerous.

TSS was discovered in the 1970s, and most of those affected were young women who used tampons

topcare® TAMPONS

INSTRUCTIONS

EN cont...

during menstruation, hence the name "tampon disease." TSS is caused by a toxin that can be produced by certain strains of Staphylococcus aureus bacteria. Many people have protective antibodies against the toxin without having had the disease, while others, lacking these protective antibodies, can become seriously ill. Most cases that occur today are unrelated to menstruation or tampons and often occur with wound infections. The disease is very rare, but it is crucial to recognize the symptoms of TSS because the mortality rate is high among patients who do not receive early treatment.

TSS SYMPTOMS

TSS causes high fever, fainting spells, dizziness, vomiting, and diarrhea. Common symptoms include red eyes, a red tongue, and red mucous membranes in the mouth. Additional symptoms include skin rash and muscle aches, as well as swelling of the palms of the hands and soles of the feet. The disease can be life-threatening, so it is important to seek immediate medical attention if you experience symptoms. Remember to inform the doctor that you use tampons. TSS can occur even if you do not use tampons, so always consult your doctor if you want to use tampons but have had symptoms of TSS. There are no preventive measures against TSS, but if you use tampons, remember to change them frequently.

Produced in EU for Creative Concept Nordic AB, SE-183 79 Täby www.creativeconcept.se